

Limitations

1. Although this medium is selective for gram negative organisms, biochemical identification and serological testing using pure cultures is recommended for complete identification.
2. It is advised to incubate for recommended period and temperature to avoid misinterpretation of results.
3. It is advised to read the results immediately after incubation, as overgrowth of *Proteus* species may mask other colonies.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Light yellow to pink homogeneous free flowing powder

Gelling

Firm, comparable with 1.5% Agar gel

Colour and Clarity of prepared medium

Orange red coloured clear to slightly opalescent gel forms in Petri plates

Reaction

Reaction of 5.15% w/v aqueous solution at 25°C. pH : 7.1±0.2

pH

6.90-7.30

Cultural Response

Cultural characteristics observed after an incubation at 35 - 37°C for 18 - 24 hours.

Cultural Response

Organism	Inoculum (CFU)	Growth	Recovery	Colour of Colony
<i>Escherichia coli</i> ATCC 25922 (00013*)	50-100	luxuriant	≥50%	pink to red with bile precipitate
<i>Yersinia enterocolitica</i> ATCC 13048 (00175*)	50-100	luxuriant	≥50%	pink to red
<i>Enterococcus faecalis</i> ATCC 29212 (00087*)	50-100	none-poor	≤10%	pale pink to red
<i>Proteus vulgaris</i> ATCC 13315	50-100	luxuriant	≥50%	colourless
<i>Salmonella Paratyphi A</i> ATCC 9150	50-100	luxuriant	≥50%	colourless
<i>Shigella flexneri</i> ATCC 12022 (00126*)	50-100	fair to good	30-40%	colourless
<i>Salmonella Paratyphi B</i> ATCC 8759	50-100	luxuriant	≥50%	colourless
<i>Salmonella Enteritidis</i> ATCC 13076 (00030*)	50-100	luxuriant	≥50%	colourless
<i>Salmonella Typhi</i> ATCC 6539	50-100	luxuriant	≥50%	colourless
<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 25923 (00034*)	≥10 ³	inhibited	0%	
<i>Corynebacterium diphtheriae</i> type <i>gravis</i>	≥10 ³	inhibited	0%	
<i>Staphylococcus epidermidis</i> ATCC 12228 (00036*)	≥10 ³	inhibited	0%	

Key : * Corresponding WDCM numbers.

Formerly known as *Enterobacter aerogenes*

Storage and Shelf Life

Store below 30°C in a tightly closed container and the prepared medium at 2 - 8°C. Use before expiry date on the label. On opening, product should be properly stored dry, after tightly capping the bottle in order to prevent lump formation due to the hygroscopic nature of the product. Improper storage of the product may lead to lump formation. Store in dry ventilated area protected from extremes of temperature and sources of ignition. Seal the container tightly after use. Use before expiry date on the label. Product performance is best if used within stated expiry period.

Please refer disclaimer Overleaf.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (6,7).

Reference

1. MacConkey, 1900, The Lancet, ii:20.
2. MacConkey, 1905, J. Hyg., 5:333.
3. Salfinger Y., and Tortorello M.L. Fifth (Ed.), 2015, Compendium of Methods for the Microbiological Examination of Foods, 5th Ed., American Public Health Association, Washington, D.C.
4. Baird R.B., Eaton A.D., and Rice E.W., (Eds.), 2015, Standard Methods for the Examination of Water and Wastewater, 23rd ed., APHA, Washington, D.C.
5. Wehr H. M. and Frank J. H., 2004, Standard Methods for the Microbiological Examination of Dairy Products, 17th Ed., APHA Inc., Washington, D.C.
6. Isenberg, H.D. Clinical Microbiology Procedures Handbook. 2nd Edition.
7. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock, D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.

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
Revision : 04/ 2018



In vitro diagnostic medical device



CE Marking



Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,
23 Vadhani Industrial Estate,
LBS Marg, Mumbai-86, MS, India



CE Partner 4U ,Esdoornlaan 13, 3951
DB Maarn The Netherlands,
www.cepartner4u.eu

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.