

collection. *Shigella* species may not grow on this medium. Prior initiation of antibiotic therapy, low urine pH (less than 5) etc. may result in low urine count from infected patients.

Type of specimen

Clinical:Urine sample

Specimen Collection and Handling

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (3,4).

Warning and Precautions

In Vitro diagnostic use. Read the label before opening the container. Wear protective gloves/protective clothing/eye protection/face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Limitations

1. This medium is recommended for urine infection. Low urine count may be a result of antibiotic therapy, low pH of urine.
- 2.Recovery depends on the urine count.
- 3.Inoculate the medium immediately after urine collection.
- 4.*Shigella* species may not grow on this medium.
5. For better results, the medium should not be incubated for more than 24 hours because if lactose fermenters predominate, the entire medium may turn pink masking the presence of non-lactose fermenters.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Light yellow to greyish yellow homogeneous free flowing powder

Gelling

Firm, comparable with 1.5% Agar gel

Colour and Clarity of prepared medium

Greenish blue clear to slightly opalescent gel forms in Petri plates

Reaction

Reaction of 3.62% w/v aqueous solution at 25°C. pH : 7.5±0.2

pH

7.30-7.70

Cultural Response

Cultural characteristics observed after an incubation at 35-37°C for 18-24 hours

Organism	Inoculum (CFU)	Growth	Recovery	Colour of colony
# <i>Klebsiella aerogenes</i> ATCC 13048 (00175*)	50-100	good-luxuriant	≥70%	greyish green, mucoid
<i>Escherichia coli</i> ATCC 25922 (00013*)	50-100	good-luxuriant	≥70%	bright pink with pink halo
<i>Enterococcus faecalis</i> ATCC 29212 (00087*)	50-100	good-luxuriant	≥70%	orange-yellow or greenish
<i>Proteus mirabilis</i> ATCC 25933	50-100	good-luxuriant	≥70%	blue-green
<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 25923 (00034*)	50-100	good-luxuriant	≥70%	golden-yellow

Streptococcus pyogenes 50-100 good-luxuriant $\geq 70\%$ greyish green
ATCC 19615

Key : *Corresponding WDCM numbers.

#- Formerly known as *Enterobacter aerogenes*

Storage and Shelf Life

Store between 10-30°C in a tightly closed container and the prepared medium at 20-30°C. Use before expiry date on the label. On opening, product should be properly stored dry, after tightly capping the bottle in order to prevent lump formation due to the hygroscopic nature of the product. Improper storage of the product may lead to lump formation. Store in dry ventilated area protected from extremes of temperature and sources of ignition Seal the container tightly after use. Use before expiry date on the label. Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (3,4).

Reference

1. Bevis T. D., 1968, J. Med. Lab. Technol., 25:38.
2. Dixson J. M. S. and Clark M. A., 1968, Conc. Med. Assoc. J., 99 (15)
3. Isenberg, H.D. Clinical Microbiology Procedures Handbook. 2nd Edition.
4. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.
5. Mackey and Sandys, 1965, Br. Med. J., 2:1286.
6. Mackey and Sandys, 1966, Br. Med. J., 1:1173.
7. Sandys, 1960, J. Med. Lab. Technol., 17:224.

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
Revision : 04/ 2018



In vitro diagnostic medical device



CE Marking



Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,
23 Vadhani Industrial Estate,
LBS Marg, Mumbai-86, MS, India



CE Partner 4U, Esdoornlaan 13, 3951
DB Maarn The Netherlands,
www.cepartner4u.eu

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.