

<i>Stenotrophomonas maltophilia</i> ATCC 13637	$\geq 10^3$	inhibited	0	0%	30 -35 °C	≥ 72 hrs
<i>Escherichia coli</i> ATCC 25922 (00013*)	$\geq 10^3$	inhibited	0	0%	30 -35 °C	≥ 72 hrs
<i>Escherichia coli</i> NCTC 9002	$\geq 10^3$	inhibited	0	0%	30 -35 °C	≥ 72 hrs
<i>Staphylococcus aureus</i> subsp.aureus ATCC 6538 (00032*)	$\geq 10^3$	inhibited	0	0%	30 -35 °C	≥ 72 hrs
<i>Staphylococcus aureus</i> subsp.aureus ATCC 25923 (00034*)	$\geq 10^3$	inhibited	0	0 %	30 -35 °C	≥ 72 hrs
<i>Salmonella</i> Typhimurium ATCC 14028 (00031*)	$\geq 10^3$	inhibited	0	0%	30 -35 °C	≥ 72 hrs
<i>Proteus mirabilis</i> ATCC 29906 (00023*)	$\geq 10^3$	inhibited	0	0%	30 -35 °C	≥ 72 hrs

Key : *Corresponding WDCM numbers.

Storage and Shelf Life

Store between 10-30°C in a tightly closed container and the prepared medium at 20-30°C. Use before expiry date on the label. On opening, product should be properly stored dry, after tightly capping the bottle in order to prevent lump formation due to the hygroscopic nature of the product. Improper storage of the product may lead to lump formation. Store in dry ventilated area protected from extremes of temperature and sources of ignition Seal the container tightly after use. Use before expiry date on the label.

Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (5,6).

Reference


- Baird R.B., Eaton A.D., and Rice E.W., (Eds.), 2015, Standard Methods for the Examination of Water and Wastewater, 23rd ed., APHA, Washington, D.C.
- Brown and Lowbury, 1965, J. Clin. Pathol., 18:752.
- Forbes B. A., Sahm A. S. and Weissfeld D. F., Bailey & Scotts Diagnostic Microbiology, 10th Ed., 1998, Mosby, Inc., St. Louis, Mo.
- Goto and Enomoto, 1970, Jpn. J. Microbiol., 14:65.
- Isenberg, H.D. Clinical Microbiology Procedures Handbook. 2nd Edition.
- Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.
- King, Ward and Raney, 1954, J. Lab. Clin. Med., 44:301.
- Lowbury, 1951, J. Clin. Pathol., 4:66.
- Lowbury and Collins, 1955, J. Clin. Pathol., 8:47
- MacFaddin J. F., 1985, Media for Isolation-Cultivation-Identification -Maintenance of Medical Bacteria, Vol. I, Williams and Wilkins, Baltimore.
- Murray P. R., Baron J. H., Pfaller M. A., Jorgensen J. H. and Tenover F. C., (Ed.), 2003, Manual of Clinical Microbiology, 8th Ed., American Society for Microbiology, Washington, D.C.
- USFDA Bacteriological Analytical Manual, 2005, 18th Ed., AOAC, Washington, DC.
- Williams, (Ed.), 2005, Official Methods of Analysis of the Association of Official Analytical Chemists, 19th Ed., AOAC, Washington, D.C.



In vitro diagnostic medical device



CE Marking

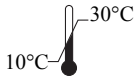

Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,
23 Vadhani Industrial Estate,
LBS Marg, Mumbai-86, MS, India



CE Partner 4U ,Esdoornlaan 13, 3951
DB Maarn The Netherlands,
www.cepartner4u.eu

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.