



## Quality Control

### Appearance

Etgc o"vq" {gmnqy"jq o qi gpgqwu"htgg"hnqy kpi"rqy fgt

### Gelling

Hkto.eqo rctcdng"ykvj"307 ' "Cict"ign

### Colour and Clarity of prepared medium

Ogfkwo"co dgt"eqnqwtgf."engct"vq"unki jvn {"qrcnguegpv"ign"ykvj"c"dnwkuj"vkpi g"hqt o u"kp"Rgytk"rncvgu0

### Reaction

Tgcevkqp"qh"8037 ' "ylx"cswgqwu"uqnwvkqp"cv"47ÅE0"r J "<"9020204

### pH

80: 2/9042

### Cultural Response

Ewnvwtcn"ejctcevgtkuvkeu"qdugt xgf"ykvj"cffgf"Dcevgtkfgu"Ugngevkgxg"Uwr rng o gpv"\*HF284+"y jgp"kpewdcvgf"cpctgdkecm {"cv 57/59ÅE"hqt"62/6: "jqwtu0

### Cultural Response

Organism	Inoculum (CFU)	Growth	Recovery	Esculin hydrolysis
<b>Cultural Response</b> <i>Bacteroides fragilis</i> ATCC 23745	72/322	iqqf/nwzwtkcpv	@?72 '	rqukvkxg tgcevkqp.dncemgpkpi qh"vjg"ogfkwo
<i>Bacteroides vulgatus</i> ATCC 8482	72/322	iqqf/nwzwtkcpv	@?72 '	pgicvkxg tgcevkqp
<i>Clostridium perfringens</i> ATCC 13124 *22229, +	@?32	kpjkdkgf	2 '	
<i>Proteus mirabilis</i> ATCC 12453	72/322	pqqg"/rqqt	>?32 '	pgicvkxg tgcevkqp

Mg {"< ", Eqttgurqpflpi "Y FEO"pw o dgtu0

### Uvqtcig"cpf"Ujgnh"Nkhg

Store between 10-30°C in a tightly closed container and the prepared medium at 2-8°C. Use before expiry date on the label. On opening, product should be properly stored dry, after tightly capping the bottle in order to prevent lump formation due to the hygroscopic nature of the product. Improper storage of the product may lead to lump formation. Store in dry ventilated area protected from extremes of temperature and sources of ignition Seal the container tightly after use. Use before expiry date on the label.

Product performance is best if used within stated expiry period.

### Fkurqucn

Wugt" o wuv" gpuwtg" uchg" fkurqucn" d{ " cwvqencxkpi" cpflqt" kpekpgtcvkqp" qh" wugf" qt" wpwuedng" rtgrctcvkqpu" qh" vjku" rtqfwew0" Hqmqy" guvednkujgf" ncdqtcvqt{ " rtqegfwtgu" kp" fkurqukpi" qh" kphgevkqu" o cvgtkcn" vjcv" eq o gu" kp vq" eqpvcev" ykvj" enpkkecn" uc o rng" o wuv" dg" fgeqpvvc o kpcvfg" cpf" fkurqugf" qh" kp" ceeqtf cpeg" ykvj" ewttgpv" ncdqtcvqt{ " vgejpkswgu" \*6.7+0"

### Reference

- 30" Owttc{ "R0" T0. "Dctgp" G0" L. "Lqt i gpugp" L0" J 0. "Rhcngt" O0" C0. " [ qnmgp T0" J 0. \*Gfu0+.": vj "Gf0. "4225. "Ocpwcn" qh" Enpkkecn" Oketqdknqi { . CUO. " Ycu jkpi vqp. " F0E0
- 40 Nkxkpi uvqp. "Mq o kpqu" cpf" [ gg. "3; 9: . "L0" Enkp0" Oketqdkn0. "9<66: 0
- 50 Ujk o cf g" M0. "Uwvngt" X0N0" cpf" Hkpgi qnf" U0O0. "3; 92. "Cr rn0" Oketqdkn0 3056735: : : ;
- 60 Hkpgi qnf" U0O0" cpf" Uwvngt" X0N0. "3; 93. "L0" Kphgev0" Fku0. "346<7780
- 70 OceHc f fkp" L0H0. "3; : 7. "Og fkc" hqt" Kuqncvkqp/ Ewvwxcvkqp/ Kf gpvkhkecvkqp/ Ockpv gpcpeg" qh" Og fken" Dcevgtkc. "Xqn0" K. " Ykmmk o u" cpf Ykmmkpu. "Dcnvk o qtgl

Tgxkukqp"<251423:



In vitro diagnostic medical device



CE Marking



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,  
B /4-6 , MIDC, Dindori, Nashik MH  
[www.himedialabs.com](http://www.himedialabs.com)



CE Partner 4U ,Esdoornlaan 13, 3951  
DB Maarn The Netherlands,  
[www.cepartner 4u.eu](http://www.cepartner 4u.eu)

### Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.