





sample must be decontaminated and disposed of in accordance with current laboratory techniques (10,11).

## Reference

- Holbrook R. and Anderson J., 1980, Can. J. Microbiol. 26(7):753-759
- Donovan K.O., 1958, J. Appl. Bacteriol, 21(1): 100.
- Mossel D.A.A., Koopman J. and Jongerius E., 1967, J. Appl. Microbiol. 15(3):650-653.
- Mortimer P.R. and McCann G., 1974, Lancet, 1043-1045.
- Bouza E., Grant S., Jordan C. et al, 1979, Arch. Ophthalmol. 97:498
- Wohlgemuth K., Kirkbride, C.A., Bicknell, E. J. and Ellis, R.P., 1972, J. Am. Vet. Med. Ass. 161:1691-1695.
- Kimball P.C., J. Clin. Pathol. 32:289
- American Public Health Association, Standard Methods for the Examination of Dairy Products, 1978, 14th Ed., Washington D.C.
- Salfinger Y., and Tortorello M.L. Fifth (Ed.), 2015, Compendium of Methods for the Microbiological Examination of Foods, 5th Ed., American Public Health Association, Washington, D.C.
- Isenberg, H.D. Clinical Microbiology Procedures Handbook. 2<sup>nd</sup> Edition.
- Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock, D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.
- Wehr H. M. and Frank J. H., 2004, Standard Methods for the Microbiological Examination of Dairy Products, 17th Ed., APHA Inc., Washington, D.C.


Revision : 02/2019



In vitro diagnostic medical device



CE Marking

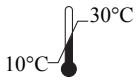

Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,  
23 Vadhani Industrial Estate,  
LBS Marg, Mumbai-86, MS, India



CE Partner 4U, Esdoornlaan 13, 3951  
DB Maarn The Netherlands,  
[www.cepartner4u.eu](http://www.cepartner4u.eu)

### Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.