



Aeromonas Isolation Medium HiVeg™ Base

MV884

Intended use

Aeromonas Isolation Medium HiVeg™ Base with added Ampicillin supplement is recommended for selective and differential isolation of *Aeromonas hydrophila* from clinical and environmental specimens.

Composition**

Ingredients	Gms / Litre
HiVeg™ special peptone	5.000
Yeast extract	3.000
L-Lysine hydrochloride	3.500
L-Arginine hydrochloride	2.000
Inositol	2.500
Lactose	1.500
Sorbose	3.000
Xylose	3.750
Synthetic detergent	3.000
Sodium thiosulphate	10.670
Sodium chloride	5.000
Ferric ammonium citrate	0.800
Bromo thymol blue	0.040
Thymol blue	0.040
Agar	12.500
Final pH (at 25°C)	8.0±0.2

**Formula adjusted, standardized to suit performance parameters

Directions

Suspend 28.15 grams in 500 ml distilled water. Heat to boiling to dissolve the medium completely. **DO NOT AUTOCLAVE**. Cool to 45-50°C and aseptically add rehydrated contents of 1 vial of Aeromonas Selective Supplement (FD039). Mix well and pour into sterile Petri plates.

Principle And Interpretation

Aeromonas species occur widely in soil and water where these species cause disease in fish and amphibians. Also found in untreated and chlorinated drinking water, raw food and raw milk (9, 10). It is observed that the major cause of gastrointestinal infections by *Aeromonas* species (10, 11) is because of ingesting infected water (12,13). This medium therefore, may be considered as a useful diagnostic aid for investigating diarrhoeal disease (5,14). Aeromonas medium was found to be superior over some other formulae for detection of *Aeromonas* species in tap water, bottled water and foods including meat, poultry, fish and seafood (6, 7, 8). Aeromonas Isolation Medium is based on the formulation of Ryan (1). It is a modification of XLD Medium, which supports the growth of *Aeromonas*, *Plesiomonas*, *Proteus*, as well as *Enterobacteriaceae* so the medium is used as universal medium in the investigation of enteric disease. This medium is prepared by completely replacing animal based peptones with vegetable peptones to avoid BSE/TSE risks. The selectivity of the medium is increased by the addition of Ampicillin (FD039). The effectiveness of Ampicillin as a selective agent has been reported by several workers (2, 3, 4, 5). It was noted that the recovery of *Aeromonas* species was very low from fresh foods of animal origin when cultivated on clinical media. Also difficulties were encountered in distinguishing the *Aeromonas hydrophila* group from the background microflora. Polumbo et.al formulated Starch Ampicillin (SA) Agar with starch hydrolysis as the differential trait and ampicillin to suppress the background microflora (15).

HiVeg™ special peptone and yeast extract provide essential nitrogenous and carbonaceous compounds, long chain amino acids, vitamins and other essential growth nutrients. The salts provide the essential minerals and electrolytes. Sodium chloride maintains osmotic equilibrium. Lactose, sorbose, inositol and xylose are sources of carbon and energy. Ampicillin, synthetic detergent and sodium thioglycollate makes the medium selective. Bromothymol blue and thymol blue acts as indicators giving the characteristic colony colour.

Type of specimen

Clinical samples - faeces ; foods ; water samples

FCM₁ X₁6b₁ X₁6ba₁ TaW₁ TaWaZ

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (19,20)

For food, follow appropriate techniques for sample collection and processing as per guidelines (16,17)

For water samples, follow appropriate techniques for sample collection, processing as per guidelines and local standards(18). After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning and Precautions :

In Vitro diagnostic Use only. Read the label before opening the container. Wear protective gloves/protective clothing/eye protection/ face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets

Notes

1. 4W₁6ba₁Y4` c\W₁_a`Y`eXh₁eW₁bef₁X₁6i X₁fb₁T₁6ba₁Y4X₁b` ba₁l₁T₁TaW₁b₁X₁` \a₁T₁X₁V₁6a₁f` \a₁Z₁Y₁b₁e₁f
2. It is advised to incubate for recommended period and temperature to avoid misinterpretation of results.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Light yellow to light tan homogeneous free flowing powder

Gelling

Firm, comparable with 1.25% Agar gel.

Colour and Clarity of prepared medium

Dark green coloured clear to slightly opalescent gel forms in Petri plates.

Reaction

Reaction of 5.63% w/v aqueous solution at 25°C. pH : 8.0±0.2

pH

7.80-8.20

Cultural Response

Cultural characteristics observed with added Aeromonas Selective Supplement (FD039) after an incubation at 35-37°C for 18-24 hours.

Cultural Response

Organism	Inoculum (CFU)	Growth	Recovery	Colony characteristics
Cultural Response				
<i>Aeromonas hydrophila</i> ATCC 7966 (00063*)	50-100	luxuriant	≥50%	dark green, opaque with dark centre
<i>Escherichia coli</i> ATCC 25922 (00013*)	≥10 ³	inhibited	0%	
<i>Pseudomonas aeruginosa</i> ATCC 27853 (00025*)	50-100	good-luxuriant	≥50%	blue/grey, translucent pinpoint
<i>Salmonella Typhi</i> ATCC 6539	≥10 ³	inhibited	0%	
<i>Shigella flexneri</i> ATCC 12022 (00126*)	≥10 ³	inhibited	0%	

Key : *Corresponding WDCM numbers.

Storage and Shelf Life

Store below 30°C in a tightly closed container and the prepared medium at 2 - 8°C. Use before expiry date on the label. On opening, product should be properly stored dry, after tightly capping the bottle in order to prevent lump formation due to the hygroscopic nature of the product. Improper storage of the product may lead to lump formation. Store in dry ventilated area protected from extremes of temperature and sources of ignition Seal the container tightly after use. Use before expiry date on the label. Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory technique (19,20)

Reference

1. Ryan N., 1985, Personal Communication.
2. Richardson C. J., Robinson J. O., Wagener L. B., Burke V. J., 1982, Antimicrob., Chemother., 9:267.
3. Mouldsdale M. T., 1983, The Lancet, 1:351.
4. Rogol M., Sechter I., Grenber L., Gerichter Ch. B., 1979, J. Med. Microbiol., 12:229.
5. Atkinson M., 1986, Culture, Vol. 7, No. 2.
6. Holmes P. and Sartory D. P., 1993, Letters in Applied Microbiol., 17: 58.
7. C. Pin M. L., Marin M. L., Garcia J. et al, 1994, Letters in Applied Microbiol., 18:190.
8. Warburton D. W., McCormick J. K., and Browen B., 1994, Can. J. Microbiol., 40:145.
9. Steering Group on the Microbiological Safety of Foods (SGMSF) in Methods for Use in Microbiological Surveillance, 1994, MAFF, Ergon House, London SWIP3TR.
10. Buchanan R. L. and Palumb S. A., 1985, J. Food Safety, 7:15.
11. Burke V. et al 1984, Appl. Environ. Microbiol., 48:361.
12. George W. L., 1987, Clin. Microbiol., Newsletter 9, 121.
13. Holmberg S. D., et al, 1986, Ann. Intern. Med., 105:683.
14. Moyer N. P., 1987, J. Clin. Microbiol., 25:2044.
15. Palumbo S. A., Maxino F., Williams A. C., Buchanan R. L., and Thayer D.W., 1985, Appl. Environ. Microbiol., 50:1027.
- 16 American Public Health Association, Standard Methods for the Examination of Dairy Products, 1978, 14th Ed., Washington D.C.
- 17 Downes F. P. and Ito K., (Ed.), 2001, Compendium of Methods for the Microbiological Examination of Foods, 4th Ed., American Public Health Association, Washington, D.C.
- 18 Greenberg A. E., Clesceri L. S. and Eaton A. D., (Eds.), 2005, Standard Methods for the Examination of Water and Wastewater, 21st ed., APHA, Washington, D.C.
- 19 Isenberg, H.D. Clinical Microbiology Procedures Handbook. 2nd Edition.
- 20 Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.


Revision : 02 / 2018



In vitro diagnostic medical device



CE Marking

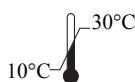

Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,
23 Vadhani Industrial Estate,
LBS Marg, Mumbai-86, MS, India



CE Partner 4U, Esdoornlaan 13, 3951
DB Maarn The Netherlands,
www.cepartner4u.eu

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.