


Are your biohazard waste bins overloaded?



Reduce plasticware consumption with

HiMedia's Hi-PCR[®] COVID-19 Detection Kit
(Dry Swab Method) -MBT189

Extraction free
RTPCR Kit

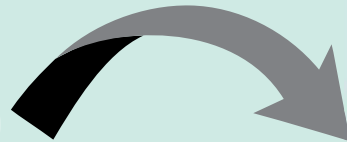

Insta Q96[®]
(LA1073/LA1074)
Real-Time PCR

Quick

Save resources

Minimum Technical
expertise

100% Sensitivity
and Specificity




Instructions for Sample Collection:




1

Carefully insert the swab provided in the kit into one nostril. The swab tip should be inserted up to 3-4cm or until resistance is met.

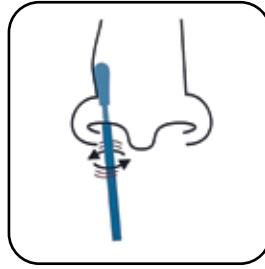

2

Roll the swab few times inside the nostril so that all the cells are collected

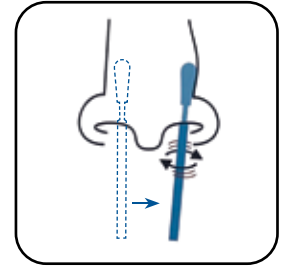

3

Repeat the same steps in the other nostril also to ensure adequate samples is collected form nasal cavities.



4

Withdraw the swab and insert it into a dry tube provided in the kit.



5


Holding the tube with one hand, break the swab with the other hand at the breakpoint.



6

Once the swab fits into the tube completely, close the tube tightly with lid. The sample can now be transported to the laboratory at Room Temperature (15°C – 25°C).

**COVID-19
sample
processed
using Dry
Swab**

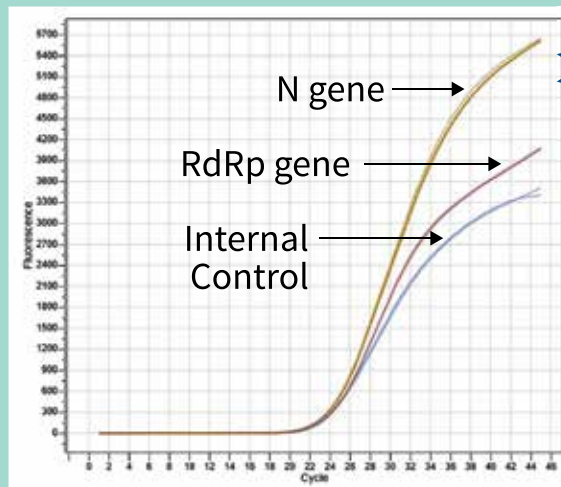

HIMEDIA®

For Life is Precious

HiMediaLaboratories™

www.himedialabs.com

CORPORATE OFFICE

Plot No. C40, Road No.21Y, MIDC, Wagle Industrial Area, Thane (West) - 400604, Maharashtra, India.

Tel : +91-22-6147 1919 / 6116 9797 / 6903 4800 | Fax : +91-22-6147 1920

Email : mb@himedialabs.com