



Technical Data

Sterile Charcoal Supplement

FD280

Recommended for growth of *Legionella* species. It is used to neutralize toxic compounds. Activated charcoal decomposes hydrogen peroxide, a metabolic product toxic to *Legionella* species.

Composition

Per vial sufficient for 500 ml medium

Ingredients

Activated Charcoal

Concentration

2g

Directions:

Rehydrate the contents of 1 vial with 10 ml of sterile distilled water to dispense the contents evenly. Aseptically add to 430 ml sterile, molten, cooled (45-50°C) Legionella Agar Base (M1845) alongwith 1 vial containing 50ml of BCYE Growth Supplement FD142. Aseptically add 10ml of sterile distilled water to bring volume to 500 ml when no selective supplement is added. Mix well and pour into sterile Petri plates. The final pH of the medium will be 6.9 ± 0.2 .

Note : After rehydrating if FD is stored for longer, slight fine precipitate may develop which may be evenly distributed before addition to the molten medium.

Type of specimen

Clinical samples - urine; Water samples

Specimen Collection and Handling

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (1,2).

For water samples, follow appropriate techniques for sample collection, processing as per guidelines and local standards (3). After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning & Precautions

In Vitro diagnostic use. For professional use only. Read the label before opening the container. Wear protective gloves/ protective clothing/eye protection/face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Storage and Shelf Life

Store at 10 - 30°C. Use before expiry date on the label.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (1,2).

Reference

1. Isenberg (Ed.), 2004, Clinical Microbiology Procedures Handbook, Vol.3, American Society for Microbiology, Washington. D.C.
2. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.
3. Baird R.B., Eaton A.D., and Rice E.W., (Eds.), 2015, Standard Methods for the Examination of Water and Wastewater, 23rd ed., APHA, Washington, D.C.

* Not For Medicinal Use

Revision : 05/ 2023



HiMedia Laboratories Pvt.
Limited, Plot No.C-40, Road
No.21Y, MIDC, Wagle Industrial
Area, Thane (W) -400604, MS,
India




CEpartner4U, Esdoornlaan 13,
3951DB Maarn, NL
www.cepartner4u.eu



In vitro diagnostic
medical device



CE Marking

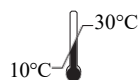

Storage temperature



**Do not use if
package is damaged**

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.