



# Technical Data

## PDTY Selective Supplement

FD334

A selective supplement for the isolation of Mycoplasma.

### Composition

Per vial sufficient for 1000 ml medium

#### \*Ingredients

Dextrose(Glucose)

Yeast extract

Thallium acetate

Penicillin G

#### Concentration

1.80grams

0.36 grams

0.036grams

3,60,000 units

### Directions:

Rehydrate the contents of one vial with 5 ml sterile distilled water and aseptically add to 1000 ml of Tryptose Serum Broth Base (Modified Newings Tryptose Broth Base) [M2019](#) along with 130ml of pig serum [RM10415](#), inactivate at 56°C for not more than 30 minutes. Mix well and distribute into sterile tubes or flasks as desired.

### Type of specimen

Clinical- nasopharyngeal and oropharyngeal swabs, pharmaceutical samples

### Specimen Collection and Handling

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (1,2). After use, contaminated materials must be sterilized by autoclaving before discarding.

### Warning & Precautions

In Vitro diagnostic use only. For professional use only. Read the label before opening the container. Wear protective gloves/ protective clothing/eye protection/face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

### Storage and Shelf Life

Store at 2 - 8°C. Use before expiry date on the label.

### Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (1,2).

### Reference

- 1.Isenberg (Ed.),2004, Clinical Microbiology Procedures Handbook, Vol.3, American Society for Microbiology, Washington. D.C.
- 2.Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.

#### \* Not For Medicinal Use


Revision : 02/2022



HiMedia Laboratories Pvt.  
Limited, Plot No.C-40, Road  
No.21Y, MIDC, Wagle Industrial  
Area, Thane (W) -400604, MS,  
India



**In vitro diagnostic  
medical device**

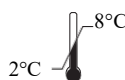

**Storage temperature**

EC | REP

CEpartner4U, Esdoornlaan 13,  
3951DB Maarn, NL  
[www.cepartner4u.eu](http://www.cepartner4u.eu)



**CE Marking**



**Do not use if  
package is damaged**

#### Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal diagnostic or therapeutic use but for laboratory, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.