



Coagulase supplement

FD362

Recommended for enumeration of Coagulase positive Staphylococci.

Composition**

Per vial sufficient for 100 ml medium

Ingredients	Concentration
Bovine fibrinogen	0.375 g
Rabbit plasma	2.500 ml
Trypsin inhibitor	2.500 mg

**Formula adjusted, standardized to suit performance parameters

Directions

Rehydrate the contents of 1 vial aseptically with 10 ml sterile distilled water. Turn the vial upside-down slowly two to three times avoiding frothing. Alternately the contents may be dissolved by gentle stirring for 1-2 minutes with a sterile glass rod or a stirrer. Immediate dissolution is not obtained. Keep it standing for 1-2 hours for better dissolution. Slight particulate matter may be observed which does not affect the performance parameter of the supplement. Aseptically add the supplement in 90 ml of sterile, molten, cooled (45-50°C) HiMRSA™ Confirmation Agar Base [M2126](#). Add the supplement slowly from the side of the flask with simultaneous gentle swirling of the flask to mix the supplement and the medium homogeneously. Mix well and pour into sterile Petri plates.

Type of specimen

Clinical- Faeces, abscess, wound samples etc; Food samples

Specimen Collection and Handling

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (1,2). For food samples, follow appropriate techniques for sample collection and processing as per guidelines (3). After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning & Precautions

In Vitro diagnostic use. For professional use only. Read the label before opening the container. Wear protective gloves/protective clothing/eye protection/face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Storage and Shelf Life

Store at 2 - 8°C. Use before expiry date on the label.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (1,2).

Reference

1. Isenberg (Ed.),2004, Clinical Microbiology Procedures Handbook, Vol.3, American Society for Microbiology, Washington. D.C.
2. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.
3. Salfinger Y., and Tortorello M.L. Fifth (Ed.), 2015, Compendium of Methods for the Microbiological Examination of Foods, 5th Ed., American Public Health Association, Washington, D.C.

* Not For Medicinal Use

Revision : 01 / 2022



HiMedia Laboratories Pvt.
Limited, Plot No.C-40, Road
No.21Y, MIDC, Wagle Industrial
Area, Thane (W) -400604, MS,
India



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
CEpartner4U, Esdoornlaan 13,
3951DB Maarn, NL
www.cepartner4u.eu



*In vitro diagnostic
medical device*



CE Marking



Storage temperature



Do not use if
package is damaged

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.