



Technical Data

Urogenital Mycoplasma Broth Base

M1374

Intended Use:

Recommended for selective cultivation of *Mycoplasma hominis* and *Ureaplasma urealyticum*.

Composition**

Ingredients	g / L
HI powder #	8.000
Tryptone	8.000
Yeast extract	4.000
Sodium chloride	3.500
Arginine hydrochloride	5.000
L-Cysteine hydrochloride	0.100
Phenol red	0.050
Final pH (at 25°C)	6.3±0.2

**Formula adjusted, standardized to suit performance parameters

Equivalent to Heart infusion powder

Directions

Suspend 14.33 grams in 425 ml purified/distilled water. Heat if necessary to dissolve the medium completely. Sterilize by autoclaving at 15 lbs pressure (121°C) for 15 minutes. Cool the medium and aseptically add rehydrated contents of 1 vial of Vitamino Growth Supplement (FD025), 1 vial of U5 Supplement (FD157), 50 ml Horse serum (RM1239) and 1 vial of PAN Selective Supplement (FD175). Mix well and dispense into tubes or flasks as desired.

Principle And Interpretation

The two *Mycoplasma* species *Mycoplasma hominis* and *Ureaplasma urealyticum*, have been implicated in urogenital infections in men and women. The organisms are also much smaller than most bacteria measuring 0.2 to 0.3 µm, hence they are able to pass through bacteriological filters. These organisms differ from other bacteria in that they lack a rigid cell wall. Individual cells are bound only by trilaminar unit membrane. Thus cultivation of *Mycoplasma* and *Ureaplasma* requires an enriched medium containing precursors for nucleic acid, protein and lipid biosynthesis. Precursors for nucleic acids and proteins are provided principally by the enriched basal peptone medium and yeast extract, while lipids are provided by the inclusion of serum. In fact, one of the principle criteria used in the taxonomic classification of these organisms is the requirement for the complex lipid cholesterol in the growth medium by certain *Mycoplasma* and *Mycoplasma*-like organisms (1). Urogenital Mycoplasma Broth Base (Mycoplasma Urogenital Broth Base) is based on the formula used by Bebear et al (2,3), Fiacco et al (4), Bonissol and Daoulas (5), Renaudin et al (6) and Bauriaud et al (7). This medium is used for selective cultivation of urogenital *Mycoplasma*, viz. *M. hominis* and *U. urealyticum* from clinical samples.

The medium contains tryptone and HI powder, which provide necessary nutrients for the growth of *Mycoplasma* and *Ureaplasma*. Yeast extract provides preformed nucleic acid precursors, necessary for the growth of fastidious *Mycoplasma*. Many *Mycoplasmas* require serum for their good growth and also presence of antibiotics (present in PAN Selective Supplement, FD175) is necessary to prevent the growth of accompanying microbial flora. Sodium chloride maintains the osmotic balance. Phenol red acts as a pH indicator. *M. hominis* metabolizes arginine to ammonia via ornithine by a three enzyme system, resulting in increase in the pH of the medium which is indicated by a colour change to red. *Ureaplasma* possess the enzyme urease and breakdown urea to ammonia indicated by a colour change to red-orange. Additional tests are required for the differentiation between *M. hominis* and *U. urealyticum*.

U. urealyticum usually causes a colour change in the broth within 24 hours except when the titre is low, the change is observed within 48 hours. *M. hominis* usually causes the colour change within 48 hours. When the titre is high, the colour change occurs within 24 hours. A negative broth should remain clear or may show a faint haze. High turbidity in the broth indicates presence of contaminants.

Type of specimen

Clinical samples - nasopharyngeal and oropharyngeal swabs

Specimen Collection and Handling

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (8,9). After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning and Precautions :

In Vitro diagnostic Use only. For professional use only. Read the label before opening the container. Wear protective gloves/protective clothing/eye protection/face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Limitations :

1. Since *Mycoplasma* species are aerobic or facultatively anaerobic but some are microaerophilic, proper incubation should be carried out for optimal recovery.
2. High turbidity in the broth indicates presence of contaminants.

Quality Control

Appearance

Light yellow to pink homogeneous free flowing powder

Colour and Clarity of prepared medium

Reddish pink coloured clear solution in tubes

Reaction

Reaction of 2.86% w/v aqueous solution at 25°C. pH : 6.3±0.2

pH

6.10-6.50

Cultural Response

Cultural response observed after an incubation at 35-37°C for 48 hours to one week with added Vitamino Growth Supplement (FD025), U5 Supplement (FD157), Horse Serum (RM1239) and PAN Supplements (FD175).

Organism	Growth	Arginine	Urea
<i>Mycoplasma hominis</i> ATCC 14027	good-luxuriant	positive reaction, red colour	negative reaction, no colour change
<i>Ureaplasma urealyticum</i> ATCC 27618	good-luxuriant	negative reaction, no red colour	positive reaction, red-orange colour

Storage and Shelf Life

Store below 10-30°C in a tightly closed container and the prepared medium at 2-8°C. Use before expiry date on the label. On opening, product should be properly stored dry, after tightly capping the bottle in order to prevent lump formation due to the hygroscopic nature of the product. Improper storage of the product may lead to lump formation. Store in dry ventilated area protected from extremes of temperature and sources of ignition. Seal the container tightly after use. Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (8,9).

Reference

1. Tully J. G., Taylor-Robinson D., 1986, *Pediatr. Infect. Dis.* 5: 5292-5295
2. Bebear C., De Barbeyrac B., Bernet C., Renaudin H., 1989, *Ann. Biol. Clin.*, 47, 415-420.
3. Bebear C., Renaudin H., Maugeim J., De Barbeyrac B., Clerc M. T., 1990, *Zentralblatt* 20, 77-82.
4. Fiaco V., Miller M. J., Carney E., Martin W. J., 1984, *J. Clin. Microbiol.*, 20, 882-865.
5. Bonissol C., Daoulas-Lebourdelle F., 1979, *Sem. Hop. Paris*, 13-14, 670-677.
6. Renaudin H., Quentin C., De Barbeyrac B., Bebear C., 1988, *Pathologie Biologie*, 36-496-499.
7. Bauriaud R., Seror C., Lareng M. B., Lefevre J. C., 1992, *Pathologie Biologie*, 40, 479-482.
8. Isenberg, H.D. *Clinical Microbiology Procedures Handbook* 2nd Edition.
9. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock, D.W. (2015) *Manual of Clinical Microbiology*, 11th Edition. Vol. 1.


Revision :06/2024




HiMedia Laboratories Pvt. Limited,
Plot No.C-40, Road No.21Y,
MIDC, Wagle Industrial Area,
Thane (W) -400604, MS, India



In vitro diagnostic
medical device

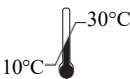

Storage temperature

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

CEpartner4U, Esdoornlaan 13,
3951DB Maarn, NL
www.cepartner4u.eu



CE Marking



Do not use if
package is damaged

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.