

# **Technical Data**

**M1446** 

## **PLET Agar Base**

## **Intended Use:**

Recommended for the selective isolation and cultivation of Bacillus anthracis.

## **Composition\*\***

Ingredients	g / L
HM infusion B from 500 g#	10.000
Tryptose	10.000
Sodium chloride	5.000
EDTA	0.300
Thallous acetate	0.040
Agar	15.000
Final pH ( at 25°C)	7.3±0.2

\*\*Formula adjusted, standardized to suit performance parameters

# Equivalent to Beef heart, infusion from

## Directions

Suspend 40.34 grams in 990 ml purified / distilled water. Heat to boiling to dissolve the medium completely. Sterilize by autoclaving at 15 lbs pressure (121°C) for 15 minutes. Cool to 45-50°C. Aseptically add rehydrated contents of 1 vial of PL Selective Supplement (FD185). Mix well and pour into sterile Petri plates.

## **Principle And Interpretation**

Anthrax is an infectious disease caused by spores of the bacterium *Bacillus anthracis*. In human anthrax, the *Bacillus* is usually demonstrable in material from a malignant pustule, sometimes in sputum from pulmonary anthrax and also in the blood in the septicemic stage of all forms of the infections. Man is relatively resistant to anthrax and laboratory workers are rarely infected. However great care should be taken to avoid escape of the long surviving spores into laboratory environment and all the procedures should be carried out in safety cabinet.

Anthrax cannot spread directly from human to human but anthrax spores can be transported by human clothings, shoes etc. In humans, anthrax is caused by exposure to dead infected animals, consumptions of infected animal tissue or exposure to light density anthrax spores from animal wool, fur, hide, etc. PLET Agar Base originally formulated by Knisley (1) is the best selective medium for cultivation of *B. anthracis* (2-4) from suspected environmental specimens, animal products or clinical specimens, inhibiting *Bacillus cereus*.

HM infusion B from solids and tryptose provide the carbonaceous and nitrogenous compounds necessary for growth whereas sodium chloride provides the osmotic equilibrium. Thallous acetate and Polymyxin (FD185) are inhibitory agents allowing growth of *B. anthracis* while inhibiting contaminants. Lysozyme (FD185) specifically suppresses the growth of gramnegative contaminants. The suspected specimen may be used directly for streaking or heat-treated or alcohol-treated specimens can be used for streaking. On incubation at 37°C for 24 hours colonies develop from 30-100% of the *B.anthracis* spores that would grow on non-selective HI Agar (M169), being smaller and smoother than on the later medium. PLET Agar Base inhibits growth of most strains of *B. cereus*, *B. subtilis*, other *Bacillus* species, *Enterobacteriaceae* and *Pseudomonas* species. Some strains of *B. cereus* from soil form colonies but they are smaller than those of *B. anthracis*, minute after 24 hours and moderately sized after 48 hours. Colonies of *B. anthracis* appear in 36-40 hours after incubation at 37°C. Roughly circular, creamy- white colonies with a ground-glass texture are further subcultured on blood agar plates for identification. Capsule production can be seen directly or on blood agar plates (2).

## **Type of specimen**

Clinical samples; Environmental specimens

## **Specimen Collection and Handling:**

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (5,6). After use, contaminated materials must be sterilized by autoclaving before discarding.

## Warning and Precautions :

In Vitro diagnostic use. For professional use only. Read the label before opening the container. Wear protective gloves/ protective clothing/eye protection/ face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

### **Limitations :**

1. Roughly circular, creamy- white colonies with a ground-glass texture are further subcultured on blood agar plates for identification. Capsule production can be seen directly or on blood agar plates (2).

#### **Performance and Evaluation**

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

## **Quality Control**

#### Appearance

Cream to yellow homogeneous free flowing powder

#### Gelling

Firm, comparable with 1.5% Agar gel

#### Colour and Clarity of prepared medium

Light amber coloured clear to slightly opalescent gel forms in Petri plates

#### Reaction

Reaction of 4.03% w/v aqueous solution at 25°C. pH : 7.3±0.2

#### pН

7.10-7.50

#### **Cultural Response**

Cultural characteristics observed with added PL Selective Supplement (FD185), after an incubation at 35-37°C for 36-40 hours.

Organism	Growth

*Bacillus anthracis* ATCC luxuriant 14578 *Bacillus cereus* ATCC 10876 inhibited

## **Storage and Shelf Life**

Store between 10-30°C in a tightly closed container and the prepared medium at 2-8°C. Use before expiry date on the label. On opening, product should be properly stored dry, after tightly capping the bottle in order to prevent lump formation due to the hygroscopic nature of the product. Improper storage of the product may lead to lump formation. Store in dry ventilated area protected from extremes of temperature and sources of ignition. Seal the container tightly after use. Product performance is best if used within stated expiry period.

#### **Disposal**

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (5,6).

#### Reference

1. Knisely R. F. 1966, J. Bacteriol, 92:784-786.

2. Murray P. R., Baron J. H., Pfaller M. A., Jorgensen J. H. and Yolken R. H., (Eds.), 2003, Manual of Clinical Microbiology, 8th Ed., American Society for Microbiology, Washington, D.C.

3. Norris J. R., Berkley C. W., Logan N. A., and ODonnell A. G., 1981, In M. P. Starr et al (Ed) The Prokaryotes: a Handbook on Habitats, Isolation and Identification of Bacteria, Vol. 2, Springer-Verlag, Berlin.

4. Parry J. M., Turnbull P. C. B. and Gibson J. R., 1983, A Colour Atlas of Bacillus species. Wolfe Medical Publications, London, United Kingdom.

5. Isenberg, H.D. Clinical Microbiology Procedures Handbook 2nd Edition.

6. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.

Revision : 02/2024





HiMedia Laboratories Pvt. Limited, Plot No.C-40, Road No.21Y, MIDC, Wagle Industrial Area, Thane (W) -400604, MS, India

CEpartner4U, Esdoornlaan 13,

3951DB Maarn, NL

www.cepartner4u.eu



IVD



-30°C Storage temperature

Do not use if package is damaged

#### Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia<sup>TM</sup> publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia<sup>TM</sup> Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.

In vitro diagnostic

medical device

HiMedia Laboratories Pvt. Ltd. Corporate Office : Plot No.C-40, Road No.21Y, MIDC, Wagle Industrial Area, Thane (W) - 400604, India. Customer care No.: 022-6147 1919 Email: techhelp@himedialabs.com Website: www.himedialabs.com