

Burkholderia cepacia Agar Plate

MP1640

Intended use

Recommended for selective isolation of *Burkholderia cepacia* from clinical sample.

Composition**

Ingredients	g/ L
Peptone	5.000
Yeast extract	4.000
Sodium pyruvate	7.000
Potassium dihydrogen phosphite	4.400
Disodium hydrogen phosphate	1.400
Bile salts	1.500
Ammonium sulphate	1.000
Magnesium sulphate	0.200
Ammonium ferrous sulphate	0.010
Phenol red	0.020
Crystal violet	0.001
Agar	12.000
PGT Selective Supplement	FD232 (2 vials)
Polymyxin B	150000IU
Gentamicin	5.0 mg
Ticarcillin	100.0 mg
Final pH (at 25°C)	6.2±0.2

**Formula adjusted, standardized to suit performance parameters

Directions

Either streak, inoculate or surface spread the test inoculum (50-100 CFU) aseptically on the plate.

Principle And Interpretation

Burkholderia cepacia is an important opportunistic pathogen and causes pulmonary infection among individuals with cystic fibrosis (CF). The organism may lead to *Burkholderia cepacia* syndrome, a neutralizing pneumonia associated with fever that culminates in to a rapid and fatal clinical deterioration (1). *B. cepacia* is difficult to isolate on routinely used laboratory media like MacConkey Agar, since *B. cepacia* is a slow grower and therefore it is usually outgrown by the faster growing *Escherichia coli*, *Staphylococcus aureus* and *Pseudomonas aeruginosa*. *Burkholderia Cepacia* Agar is based on PC medium, which was originally devised by Gilligan (2). This medium was found to be superior to MacConkey Agar for growth of *B. cepacia*. The medium is made selective for *B. cepacia* by the incorporation of bile salts, crystal violet and antibiotics. The antibiotics included are Polymyxin B, Gentamicin, Ticarcillin in the form of freeze dried supplement (FD). Peptone and yeast extract in the medium provides the nitrogenous, vitamin B source and other essential nutrients. Crystal violet, bile salts and antimicrobial agents are used as selective agents. Crystal violet and bile salts inhibits gram-positive cocci including Enterococci and Staphylococci. The antibiotics (FD) namely ticarcillin, polymyxin B and gentamicin inhibit gram-negative bacteria. *B. cepacia* metabolizes pyruvate forming alkaline end products. These end products elevate the pH of the medium. The phenol red indicator changes colour from pink orange to pink red in alkaline pH. Inoculate the plate with the specimen so as to obtain isolated colonies. The plates should be incubated for a period of 4 days to allow *B. cepacia* to grow and form colonies and subsequent colour change (2,3). The medium is not selective only for *B. cepacia*. Other organisms forming similar colonies may also grow on this medium. Therefore results obtained on this media should not be the sole criteria for identification of *B. cepacia* (4).

Type of specimen

Clinical samples - Throat swab, Sterile body fluids

Specimen Collection and Handling:

For clinical samples follow appropriate techniques for handling for handling specimens specimens as as per established guidelines (4,5). After use, contaminated materials must must be sterilized by autoclaving before discarding.

Warning and Precautions :

In Vitro diagnostic Use. For professional use only. Read the label before opening the pack. Wear protective gloves/ protective clothing/eye protection/ face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Limitations :

1. Individual organisms differ in their growth requirement and may show variable growth patterns on the medium
2. Each lot of the medium has been tested for the organisms specified on the COA. It is recommended to users to validate the medium for any specific microorganism other than mentioned in the COA based on the user's unique requirement.
3. Further biochemical tests must be carried out for confirmation.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within expiry period when stored at the recommended temperature.

Quality Control

Appearance

Sterile Burkholderia cepacia Agar in 90 mm disposable plates with smooth surface and absence of black particles/cracks/ bubbles

Colour of medium

Light Orange coloured medium

Quantity of medium

25ml of medium in 90mm disposable plate

pH

6.00 - 6.40

Sterility Check

Passes release criteria

Cultural response

Cultural characteristics observed after an incubation at 35-37°C for 48-72 hours.

Organism	Inoculum (CFU)	Growth	Recovery	Colour of colony
<i>Burkholderia cenocepacia</i> ATCC 25608	50-100	good-luxuriant	>=50%	sage green colonies with bright pink medium
<i>Pseudomonas paraeruginosa</i> ATCC 9027 (00026*)	>=10 ³	inhibited	0%	

Key : (*) Corresponding WDCM numbers,

(^) Formerly known as *Pseudomonas paraeruginosa*

(\$)Formerly known as *Burkholderia cenocepacia*

Storage and Shelf Life

On receipt store between 2-8°C. Use before expiry date on the label. Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (5,6).

Reference

1. Whitby P. W., 1998, J. Clin. Microbiol., 36:1642 1645
2. Gilligan, 1996. Clin. Microbiol. Newsl. 18:8
3. MacDonald Gilligan, Welch, Reller and Menegus, 1994, Vol. 5:1, Cystic Fibrosis Foundation, Washington,DC.
4. Christensen et al, 1980, J. Clin. Microbiol., 27:27
5. Gilligar, Gage, Bradshaw, schidlow and Deciscoco, 1985, J. Clin. Microbiol., 22
6. Isenberg, H.D. Clinical Microbiology Procedures Handbook. 2nd Edition.
7. Jorgensen, J.H., Pfaller , M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W.(2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.

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
Revision : 04/ 2024



HiMedia Laboratories Pvt. Limited,
Plot No.C-40, Road No.21Y,
MIDC, Wagle Industrial Area,
Thane (W) -400604, MS, India



IVD *In vitro* diagnostic
medical device



Storage temperature



CEpartner4U, Esdoornlaan 13,
3951DB Maarn, NL
www.cepartner4u.eu



CE Marking



**Do not use if
package is damaged**

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.