

HiCrome™ MeReSa Agar Plate

MP1674

Intended Use

For isolation and selective identification of Methicillin Resistant *Staphylococcus aureus* (MRSA) from clinical isolates.

Composition**

Ingredients	g / L
Tryptone	13.000
Yeast extract	2.500
HM Peptone B #	2.500
Sodium chloride	40.000
Sodium pyruvate	5.000
Chromogenic mixture	5.300
Agar	15.000
MeRS Selective Supplement (FD229) - 2 vials	
Methicillin (2.0 mgX2)	4.000mg
CF Selective Supplement II (FD259) - 2 vials	
Cefoxitin (3.0 mgX2)	6.000mg
Final pH (at 25°C)	7.0 ± 0.2

**Formula adjusted, standardized to suit performance parameters

Equivalent to Beef extract

Directions

Either streak, inoculate or surface spread the test inoculum (50-100 CFU) aseptically on the plate.

Principle And Interpretation

Staphylococcus aureus is an invasive pathogen that can cause disease in almost any tissue or organ in the human body, primarily in compromised individuals (1). Staphylococcal infections were earlier treated using Penicillin. But over the years resistance to this drug developed. Methicillin was the next drug of choice. While methicillin is very effective in treating most Staphylococcus infections some strains have developed resistance to methicillin and can no longer be killed by this antibiotic. These resistant bacteria are called Methicillin Resistant *Staphylococcus aureus* (MRSA) (2). Patients with breaks in their skin due to wounds, indwelling catheters or burns are those with certain risk of developing MRSA infection (3). Spread of MRSA infections can be controlled to a great extent by maintaining personal hygiene after interaction with an MRSA infected person (2).

Tryptone, HM Peptone B and yeast extract provide the essential nutrients along with carbonaceous, nitrogenous compounds, and Vitamin B complex. The chromogenic mixture incorporated in the medium is specifically cleaved by *Staphylococcus aureus* to give bluish green coloured colonies. Sodium pyruvate enhances the growth of Staphylococcus species. Sodium chloride in the medium helps to maintain the osmotic equilibrium of the medium.

High concentration of sodium chloride also helps in inhibiting the accompanying microflora. The medium is made selective for MRSA by the addition of MeRS Selective Supplement (FD229) and CF Selective Supplement II (FD259).

Type of specimen

Clinical samples: Tissue samples, wound swab, nasal swab.

Specimen Collection and Handling:

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (4,5). After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning and Precautions :

In Vitro diagnostic Use. For professional use only. Read the label before opening the pack. Wear protective gloves/ protective clothing/eye protection/ face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Limitations :

1. Some intermediate strains may show poor growth due to nutritional variations and resistance to methicillin /cefoxitin.
2. Slight colour variation may be observed depending upon the utilization of the substrate by the organism.
3. Other methicillin resistant *Staphylococcus* species may grow. Further biochemical tests must be carried out to differentiate between resistant strains.
4. Individual organisms differ in their growth requirement and may show variable growth patterns on the medium.
5. Each lot of the medium has been tested for the organisms specified on the COA. It is recommended to users to validate the medium for any specific microorganism other than mentioned in the COA based on the user's unique requirement.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Sterile HiCrome™ MeReSa Agar in 90mm disposable plates with smooth surface and absence of black particles/cracks/bubbles

Colour

Light yellow coloured medium

Quantity of medium

25ml of medium in disposable plate

pH

6.80- 7.20

Sterility Check

Passes release criteria

Cultural Response

Cultural characteristics observed after an incubation at 35-37°C for 18-48 hours.

Organism	Inoculum (CFU)	Growth	Recovery	Colour of Colony
<i>Escherichia coli</i> ATCC 25922 (00013*)	$\geq 10^3$	Inhibited	0%	
<i>Enterococcus faecalis</i> ATCC 29212 (00087*)	$\geq 10^3$	Inhibited	0%	
<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 25923 (00034*)	$\geq 10^3$	Inhibited	0%	
<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 6538 (00032*)	$\geq 10^3$	Inhibited	0%	
<i>Staphylococcus aureus</i> , MRSA ATCC 43300	50-100	Luxuriant	$\geq 50\%$	bluish green
<i>Staphylococcus epidermidis</i> ATCC 12228 (00036*)	$\geq 10^3$	Inhibited	0%	
<i>Staphylococcus xylosum</i> ATCC 29971	$\geq 10^3$	Inhibited	0%	

Key : (*) Corresponding WDCM numbers.

Storage and Shelf Life

On receipt store between 2-8°C. Use before expiry date on the label. Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (4,5).

Reference

1. DWorkin M et. al 2006. The Prokaryotes (a Handbook on the Biology of Bacteria) 3rd ed, Vol. 2, page 345.
2. Methicillin Resistant *Staphylococcus aureus* Copyright © 1997-2005 Canadian Centre for Occupational Health and Safety, Sept 19th, 2005.
3. Dr. Alan Johnson, methicillin resistant *Staphylococcus aureus* (MRSA) infection. The Support group for MSRA sufferers and Dependents, Aug 1st , 2005.
4. Isenberg, H.D. Clinical Microbiology Procedures Handbook 2nd Edition.
5. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.

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
Revision : 07/2024



HiMedia Laboratories Pvt. Limited,
Plot No.C-40, Road No.21Y,
MIDC, Wagle Industrial Area,
Thane (W) -400604, MS, India



**In vitro diagnostic
medical device**



Storage temperature



CEpartner4U, Esdoornlaan 13,
3951DB Maarn, NL
www.cepartner4u.eu



CE Marking



**Do not use if
package is damaged**

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.