

HiCrome™ ESBL Agar Plate

MP1829

Intended Use:

Recommended for the detection of Extended Spectrum β -Lactamase-producing organisms from clinical & environmental samples.

Composition**

Ingredients	g / L
Peptone mix	12.000
Chromogenic mixture	4.000
Sodium chloride	5.000
Buffer mix	4.000
Agar	15.000
AC3F Selective Supplement (FD278)	2 vials
Ceftazidime (2x1.50mg)	3.000 mg
Cefotaxime (2x1.50mg)	3.000 mg
Ceftriazone (2x1.00mg)	2.000 mg
Aztreonam (2x1.00 mg)	2.000 mg
Fluconazole (2x 5.00 mg)	10.0000 mg
Final pH (at 25°C)	6.8±0.2

**Formula adjusted, standardized to suit performance parameters

Directions

Either streak, inoculate or surface spread the test inoculum (50-100 CFU) aseptically on the plate.

Principle And Interpretation

Extended-spectrum β -lactamase (ESBL)-producing organisms are an increasing challenge for healthcare practitioners fighting healthcare-associated infections (HAIs). *Escherichia coli*, *Klebsiella pneumoniae*, and *Klebsiella oxytoca* are the most common ESBL-producing pathogens (1). ESBL-producing organisms are generally resistant to many classes of antibiotics, including aminoglycosides and fluoroquinolones; ESBL-producing organisms are able to attack newer cepheims and monobactams as well as narrow-spectrum cephalosporins and antigram-negative penicillins (2). They are associated with increased mortality and are difficult to detect and treat. The widespread use of extended-spectrum, third-generation cephalosporins, introduced in the 1980's to treat antibiotic-resistant bacteria, is believed to be a major contributor to the emergence of ESBL-producing organisms.

HiCrome™ ESBL Agar Base is chromogenic screening medium for the selective isolation of ESBL producing organisms. It contains peptone mix which serves as the carbon and nitrogen sources, long chain amino acids, vitamins and other growth nutrients. Chromogenic mixture is used to differentiate the ESBL producing organisms on the basis of colour. AC3F Selective Supplement (FD278) helps in inhibition of other contaminating organisms. ESBL producing *E.coli* grow as either pink or purple colonies.

ESBL producing members of the KESC group produce bluish green colonies; *Proteus*, *Morganella* and *Providencia* do not utilize any chromogen resulting in colourless to light brown colonies.

This medium can be inoculated with liquid suspension equivalent to 0.5 McFarland turbidity, prepared from rectal screening swabs, faecal samples or from isolated colony. Isolated colonies should not be directly plated on to this medium, because the high level inoculum may cause false positive results. Further confirmation using biochemical identification tests is recommended.

Type of specimen

Clinical samples - rectal screening swabs, faecal samples or from isolated colony.

Specimen Collection and Handling

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (1,3).

After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning and Precautions

In Vitro diagnostic Use only. For professional use only. Read the label before opening the pack. Wear protective gloves/protective clothing/eye protection/ face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Limitations

1. Individual organisms differ in their growth requirement and may show variable growth patterns on the medium.
2. Slight colour variation may be observed depending upon strains.
3. Isolated colonies should not be directly plated on to this medium, because the high level inoculum may cause false positive results.
4. Further confirmation using biochemical identification tests is recommended.
5. Each lot of the medium has been tested for the organisms specified on the COA. It is recommended to users to validate the medium for any specific microorganism other than mentioned in the COA based on the user's unique requirement.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Sterile HiCrome™ ESBL Agar in 90 mm disposable plates with smooth surface and absence of black particles/cracks/bubbles.

Colour of medium

Yellow coloured medium

Quantity of medium

25ml of medium in 90mm plate

pH

6.60-7.00

Sterility Check

Passes release criteria

Cultural Response

Cultural characteristics observed after an incubation at 35-37°C for 18-24 hours.

Organism	Inoculum (CFU)	Growth	Colour of Colony
<i>Escherichia coli</i> NCTC 13351	50-100	luxuriant	pink to purple
<i>Klebsiella quasipneumoniae</i> ATCC 700603	50-100	luxuriant	bluish green
<i>Enterobacter cloacae</i> ATCC 23355	$\geq 10^3$	inhibited	-
<i>Citrobacter freundii</i> ATCC 8090	$\geq 10^3$	inhibited	
<i>Candida albicans</i> ATCC 10231 (00054*)	$\geq 10^3$	inhibited	

Key : (*) Corresponding WDCM numbers,

(\$) Formerly known as *Klebsiella pneumoniae*.

Storage and Shelf Life

On receipt store between 2-8°C. Use before expiry date on the label. Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (1,3).

Reference

1. Isenberg, H.D. Clinical Microbiology Procedures Handbook 2nd Edition.
2. Journal of Clinical Microbiology, February 2007, Page 501-505, Vol. 45, No. 2
3. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.

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
Revision : 02/ 2024



HiMedia Laboratories Pvt. Limited,
Plot No.C-40, Road No.21Y,
MIDC, Wagle Industrial Area,
Thane (W) -400604, MS, India



IVD *In vitro* diagnostic
medical device



Storage temperature



CEpartner4U, Esdoornlaan 13,
3951DB Maarn, NL
www.cepartner4u.eu



CE Marking



**Do not use if
package is damaged**

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.