

Hiculture™ Transport Swabs w/ Chlamyospore Medium in polystyrene tube

MS113

Recommended for recovery of *Candida albicans* from clinical specimens.

Principle :

Chlamyospore medium is prepared according to the formula of Nickerson and Mankowski (1). The medium is chemically defined, semisolid supporting the survival of *Candida* species for a sufficiently long period. Sterile cotton swabs allow absorption of specimen material while polystyrene shaft allows semiflexibility to the swab stick, aiding in collection.

Description :

Each polystyrene tube contains sterile chlamyospore transport medium. Cotton swabs on polystyrene shaft is provided separately.

Quality Control :

Appearance:

Sterile Chlamyospore Medium in tubes w/ sterile cotton swabs.

Colour: Blue coloured medium.

Quantity: 8 ml of medium in tubes.

Reaction: Reaction pH of medium is 5.1 ± 0.2 .

Sterility Testing :

Sterility testing was carried out by inserting sterile swab in sterile Chlamyospore Medium and then streaking on sterile Soyabean Casein Digest Agar (M290)

Result :

After incubation at 35 - 37°C upto 14 days.

Medium	Growth
Soyabean Casein Digest Agar	No growth

**Hiculture™ Transport Swabs w/ Chlamydospore Medium
in polystyrene tube****MS113**

Viability of *Candida* species viz. *Candida albicans* (ATCC 10231), *Candida kruisii* (ATCC 24408), *Candida tropicalis* (ATCC 1369), *Candida minosa* was established for a period upto 48 hours. Organisms grew luxuriantly and chlamydospore formation was observed when inoculated and recovered on respective media like Sabouraud Dextrose Agar (M063) and incubated at 22-28⁰C for 48 hours.

Storage and Shelflife :

On receipt store between 2 – 8°C with caps firmly screwed. DO NOT FREEZE. Use before expiry date on the label.

Reference :

1. Nickerson and Mankowski, 1953, J.Inf. Dis, 92:20.

**Disclaimer :**

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.